

# Useful Handling Tips for Ecoflac® Plus Container

## Practice Suggestion for Secondary Infusion

### 1 REMOVE SEALING FOIL



Remove one of the tamper-evident sealing foils from the Ecoflac Plus TwinCap. The surface below the foil is sterile, it is not required to disinfect prior to use. Both ports are identical and can be used irrespectively for preparation or infusion.

### 2 SPIKING THE CONTAINER FOR USE



While the container is standing, insert the infusion set spike into the open port at a straight 90° angle without rotation.

### 3 HANG AND PRIME



Raise the IV pole to ensure there is 8-12" between the primary container and Ecoflac Plus. After priming, close the roller clamp and connect the secondary infusion set to the upper Y site of the primary infusion set. Connect the primary IV set to the patient access. Open roller clamps and adjust flow.

### 5 DURING INFUSION



To ensure complete delivery of medication the Ecoflac Plus container must remain vertical while infusing.

### 4 PREPARING FOR SECONDARY INFUSION



To ensure that the container properly collapses, the secondary medication set vent must be opened. To prevent concurrent sympathetic flow you should also clamp off the primary line. Always confirm the secondary line is running by visually inspecting the drip chamber.

### 6 DISPOSE USED PRODUCTS



At the end of the infusion, dispose of used products as per facility protocol.