

Ecoflac® Plus Container

Helpful Hints Page 1 of 2

Refer to product labeling for full prescribing information.

Ecoflac Plus containers are available in 50 mL (partially filled in a 100mL Ecoflac Plus container, 20 per box) and 100 mL (20 per box). The cardboard transport box has an easy opening lid made with perforated lines in the box to facilitate light protected storage of the Ecoflac Plus containers.

TWINCAP® DOUBLE PORT SYSTEM

The Ecoflac Plus container has a Twincap double port system that can also be used as a single port system. Both ports are identical for the use of medication admixture or spiking for infusion. Each port is covered by aluminum foil to protect against microbial contamination. Two resealing puncture sites are **kept germ-free** by two separate sealant foils. Therefore there is no need for disinfection of the ports prior to use. (The Ecoflac Plus container port is not made with natural rubber latex.) The **large ring** around the port neck helps to reduce the risk of needlestick injuries upon puncture of the port with a needle.

INFUSION ADMINISTRATION



REMOVE SEALING FOIL

Step 1: Remove one of the tamper-evident sealing foils on the Twincap post. The surface below the foil is germ-free, it is not required to disinfect prior to use. Both ports are identical and can be used irrespectively for drug preparation or infusion.



INSERT SPIKE

Step 2: When inserting an infusion set, grip the neck of the Ecoflac Plus container to stabilize the container.

Step 3: Always insert the infusion set straight, not at an angle. Avoid rotating the spike when piercing the port. When the set is removed, the port on Ecoflac Plus reseals after several seconds.

Step 4: During infusion, the Ecoflac Plus container collapses completely without any need for ventilation.

Step 5: Please ensure that the Ecoflac Plus Container remains vertical while infusing.

Ecoflac® Plus Container

Helpful Hints Page 2 of 2

SHIPPING INSTRUCTIONS

- Do not stack individual cases more than 5 high
- Do not mix containers sizes in a stack (all 50mL, all 100mL, etc.)
- Avoid rough handling; do not toss cases while loading and unloading
- Ensure cases are stacked squarely on top of each other
- Avoid freezing and excessive heat
- Maintain upright orientation of cases

STACKING ECOFLAC PLUS IN BINS

Ecoflac Plus can be stored in both a vertical and horizontal position. The transport box for Ecoflac Plus is designed to store Ecoflac Plus containers in an easily accessible fashion while protecting the containers from sunlight. Open the Ecoflac Plus container along the perforated lines of the transport box.

