

Mini-Plasco® Container

Helpful Hints Page 1 of 1

Refer to product labeling for full prescribing information.

Mini-Plasco containers are available in 10 mL (20 per case). The containers are not packaged with an overwrap. The cardboard transport box has an easy opening lid made of perforated lines in the box to facilitate light protected storage of the Mini-Plasco containers.

Mini-Plasco's luer connector and circular corpus of the ampule opening allows for a direct needlefree connection of a syringe.


WITHDRAWAL OF SOLUTION

Step 1: Holding the block of ampoules in both hands, use one hand to twist and break away a single ampoule. The neighbouring ampoules remain closed.

Step 2: The ampoule can be opened easily by twisting the twist-off cap. The integrated finger stopper enables a secure grip and prevents contamination of the ampoule opening.

Step 3: Support syringe insertion with the pads of your fingers to ensure aseptic technique and maintain sterility. Tightly connect a Luer or Luer-Lock syringe. No needle is required.

Step 4: Mini-Plasco connect collapses during withdrawal. This allows for vent-free aspiration in one single step. Mini-Plasco is designed to minimize exposure of the solution or drug to the environmental air.

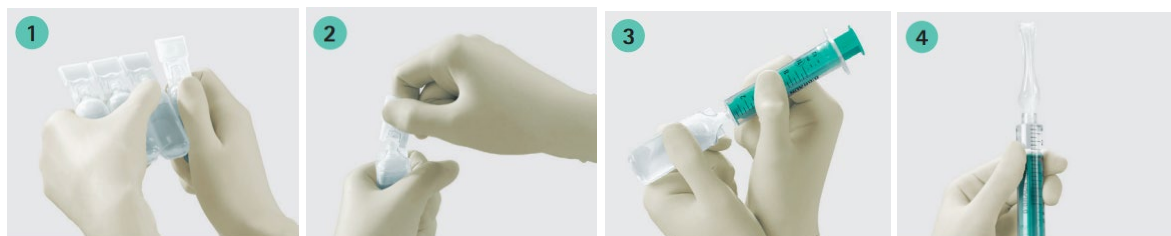

SHIPPING INSTRUCTIONS

- Avoid rough handling
- Ensure packs are stacked squarely on top of each other
- Avoid freezing and excessive heat
- Maintain upright orientation of packs

MINI-PLASCO STORAGE

Mini-Plasco can be stored in both a vertical and horizontal position. The transport box for Mini-Plasco is designed to store Mini-Plasco containers in an easily accessible fashion while protecting the containers from sunlight. Open the Mini-Plasco container along the perforated lines of the transport box.