

Easypump[®] ST/LT

Patient Guide

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Elastomeric Infusion Pump System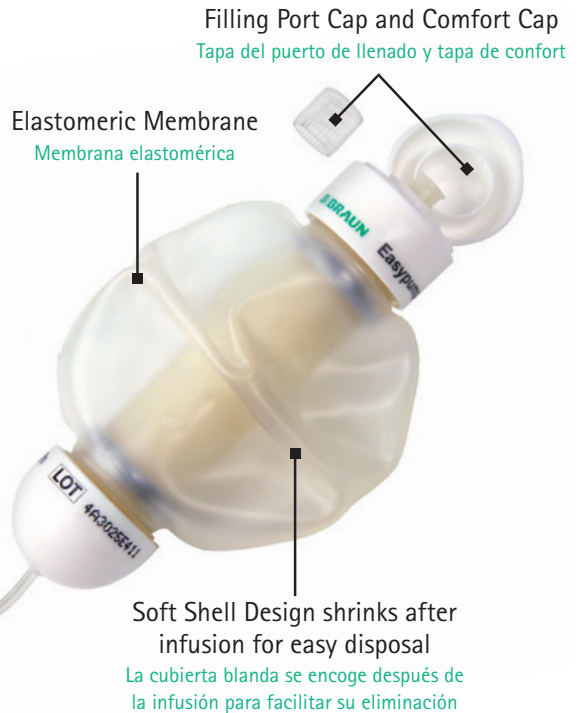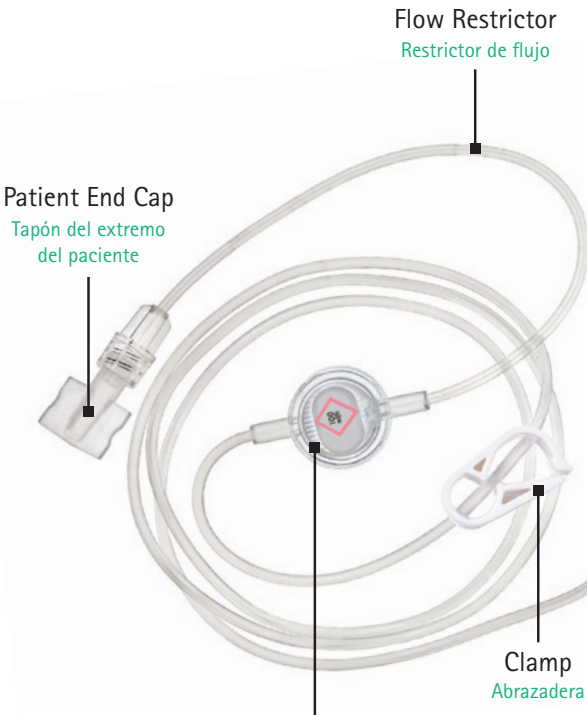

The Easypump® ST/LT disposable infusion system is designed to provide safe and convenient delivery of intravenous medications. Easypump ST is designed for short-term infusions (30 minutes up to 4 hours). Easypump LT is designed for long-term infusions (1-5 days). This guide is provided to help you use the Easypump ST/LT infusion system safely and correctly. Please be sure to read and understand all the instructions below prior to beginning your infusion.

Your medication is called _____

You will be receiving _____ dose(s) daily

at the following time(s): _____

Your healthcare provider is responsible for instructing you in the use of this infusion system. If you have any questions about your Vascular Access Device, your therapy, or the Easypump ST/LT device, call your home healthcare provider at:

() _____ - _____

STORAGE:

Your Easypump® ST/LT devices should be stored: _____ In the refrigerator
_____ Other _____

CAUTION: The Easypump ST/LT device must be allowed to reach room temperature prior to use.

If the solution is colder than room temperature, the infusion time will likely be longer than anticipated. Failure to allow solution to reach room temperature can lead to longer infusion time. It should be near room temperature when you use it. Simply remove each dose from the refrigerator as prescribed before you plan to infuse (typically 6-12 hours).

WARNING: DO NOT HEAT the Easypump ST/LT device in a microwave oven, direct sunlight, hot water or any other heat source. Heating may cause problems with the device and flow rate.

PREPARING FOR THE INFUSION

- ▶ Wash your hands thoroughly, as instructed by your healthcare provider.
- ▶ Check the drug label on your Easypump ST/LT device. Be sure that your name, the medication name and the dosage are correct. Be sure that the expiration date has not passed.
- ▶ Be sure that the clamp on the tubing is in the closed position.

► In a clean work area described by your healthcare provider, gather the supplies you will need:

- Easypump® ST/LT device
- Alcohol (or other antiseptic) prep pad(s)
- Flush syringes (Your healthcare provider will provide information about how to flush your Vascular Access Device before and after your infusion.)

1 CONNECTING YOUR Easypump ST/LT DEVICE

- Always swab the septum on the end of your vascular access device with an alcohol (or other antiseptic) prep pad and allow to dry completely.
- Remove the Patient End Cap from the Luer tip end (the end with the screw-like edges) of the Easypump ST/LT tubing. Avoid touching the open end.
- Insert the Luer tip end of the tubing into the valve septum of your vascular access device. Turn the luer-lock clockwise to secure the connection.
- Do not over tighten.
- Tape flow restrictor to the skin.

2 STARTING YOUR INFUSION


- ▶ Open the clamp to begin the flow of fluid.
- ▶ The Easypump® ST/LT device does not need to be positioned above your vascular access device.
- ▶ Do not squeeze the pump. The fluid will flow by itself.

3 DURING YOUR INFUSION

- ▶ Place the Easypump ST/LT device in a carrying pouch, in your pocket or on a nearby surface while receiving your medication.
- ▶ If you are sleeping during the infusion, keep the pump above the covers and do not place the pump on the floor or hang the pump on the bedpost.

Do not use while bathing, showering, or swimming.

4 ENDING YOUR INFUSION

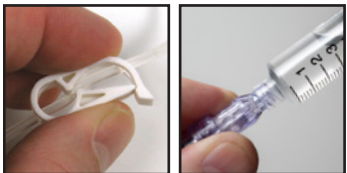- ▶ All your medication has been delivered when the pump is empty and deflated. You will be able to feel the hard inner core.
- ▶ Close the clamp. Disconnect the luer tip from the valve septum of your vascular access device and disconnect the pump from your catheter.
- ▶ Flush your catheter as instructed by your healthcare provider.
- ▶ Dispose of pump and supplies as instructed by your healthcare provider.

5 TROUBLESHOOTING

- ▶ If pump does not seem to be working, make sure:
 - The pump is at room temperature.
 - The white clamp on the tubing is open (moves freely on the tubing).
 - All clamps on your catheter are open.
 - There are no kinks in the pump tubing.
 - Follow your healthcare provider's instructions to make sure that your catheter is working properly.

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